99.99 mm

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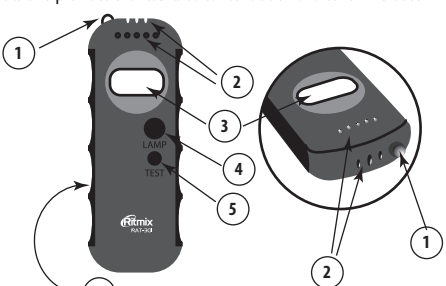
User manual 

Digital breath alcohol tester Ritmix RAT-301

Function: Breathalyzer is designed for domestic use to measure the concentration of alcohol vapor in the breath.

Thanks for purchasing Ritmix RAT-301 hand held Digital Alcohol Analyzer. Ritmix RAT-301 Alcohol tester is for making the personal test of measuring the alcohol in the exhaled vapor. Tester recalculates the data and provides the results as concentration of alcohol in blood.


- 1 Torch
- 2 Breath Inhaler
- 3 Digital display
- 4 Torch Switch
- 5 Power button
- 6 Battery compartment

Features

1. Digital display and sound signals
2. Quick response and resume.
3. Torch function, with low power consumption
4. Hand strap
5. Use two "AAA" alkaline batteries

Battery installation

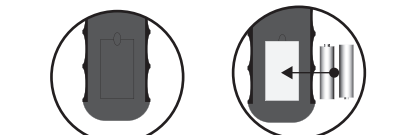1. Slide the battery compartment to open lid from the back of the unit.

2. Insert two "AAA" size alkaline batteries with correct polarity as indicated.
3. Replace the battery compartment lid.
4. Change the batteries when the alcohol tester doesn't have enough power to turn on or the torch and the torch becomes dim or you see «LO» message on the screen. Change the batteries only on the new one, which were not used before and change the both batteries at once. During the long time storage without using the device, don't keep the batteries inside.

How to use the alcohol tester
Warming up the sensor.

1. Press and hold the power button (5) to turn on the alcohol tester during 1 sec, after that you will hear the sound signal and the LCD display will turn on.
2. The unit enters into warm-up mode. «Warm up» words will be shown on the display and it starts count down from 10 to 00.

Note:
 * It always has a large number of gas & impurity adsorbed on the surface of alcohol sensor, when first time taking out the tester from the packaging, or after long time no using for the tester, it need more time for cleaning and warm up the sensor, you should repress the power button and retry several times if possible.

Measurement

1. The "Blow" symbol is displayed when warm-up is succeeding, and the 10 to 00 countdown is started that indicate the tester is in testing mode. After finishing of the warm-up process the display will show "Blow"

Blow into the breath inhaler for 10 seconds with normal power after a deep inhalation, till you don't hear the sound signal. Read the test result from the alcohol tester LCD. After testing you will hear the beep signal and results of the test will shown on the LCD.

2. There are 3 levels of the alcohol concentration shown on the monitor:
 - Less than 0.1 ppm
 - 0.1-0.2 ppm
 - More than 0.2 ppm.The "Caution" symbol is displayed if the alcohol concentration is the level:

3. Results will be shown on the monitor during 10 sec.

Note
 If you didn't blow into the breath inhaler during when the tester was in the testing mode, than LCD screen will show 0.

To obtain more accurate result, manufacturer recommends you to check repeatedly 2 or 3 times.

Don't use the alcohol tester if there is a strong wind or there is dust. If the cigarette smoke or saliva will fall on tester, it might cause the damage of the alcohol sensor.

Keep the alcohol tester in a dry and clean place at a temperature -5...+40°С.

Please do not dismantle and modify this device. Please use this device according to the user manual.

Troubleshooting

If the alcohol tester provides the wrong results, than please blow into the sensor with the dry warm air for few minutes.

Auto power off

After the result is display for about 10 seconds, the "OFF" symbol will appear for 2-3 seconds and the alcohol tester be turned off.

Specifications

- Alcohol tester is for making the personal test of measuring the alcohol in the exhaled vapor. Tester recalculates the data and provides the results as concentration of alcohol in blood.
- Sensor: advanced semiconductor oxide alcohol sensor

- Range: 0.0‰-1.9‰ (0.00%-0.19% ВАС, 0.0-1.9 g/l ВАС, or 0.0-0.91 mg/l* ВРАС)
- Accuracy: +/-0.01% ВАС (0.1 g/l)
- Power input: 2*1.5 V "AAA" alkaline battery (not included), DC.
- Operating time: warm up to 10 s
- Response time: up to 5 s
- Auto power off
- Dimension : 97*37*20 mm
- Environmental temperature: -5°С...+40°С

Delivery includes: hand strap, user manual, alcohol tester, built in torch.

Troubleshooting

If the alcohol tester provides the wrong results, than please blow into the sensor with the dry warm air for few minutes.

How to use the torch

Press and hold the button (4) to use the torch.

Attention!

If you didn't use the alcohol tester for a long time, before using it, please follow the recommendations of the first claus, please repeat it several times.

Precaution

- After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.

- For accurate result, do not repeat next testing within 3 minutes after last time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly. For cleaning the mouth piece please use the dry material or paper napkin.
- Do not blowing liquid into the breath pipe.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Each individual's body responds differently to alcohol. Drink one glass of wine somebody can feel slightly drunken, somebody can feel heavily drunken.
- Diabetes or low-calorie diet or other factors are leading to the rising of the concentration of ketones in exhalation and can cause false positive results.
- Any of Alcohol testers doesn't give the 100% of guarantee of the accurate results. The most accurate results give blood measuring test. That's why if the road patrol has stopped you, and alcohol tester showed the presence of alcohol in blood, require the blood test.
- A report of Worlds Health Organization shows that if the alcohol concentration reaches the 0.08 g/ 100 ml, than the risk of road accidents increases in 2.5 times to compare if there is no alcohol in blood. If the alcohol concentration in blood reaches 0.1 g/ 100 ml, risk of road accidents increases in 4.7 times, than there is no alcohol in blood. Even if the slight quantity of alcohol is drunk it increases risk of road accidents in 2 times.
- The unit is designed to be used as an aid to promote reasonable

driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility.

- Test results of alcohol tester are not probative.
- Don't drive in a drunken condition of any degree of intoxication.

Important notes

This device can be used only for personal use. The testing results of this device don't have any validity and can't be used in court. Producer and the seller doesn't have any responsibility if there will be the difference in the testing results between which were made by the third party or by other equipment. By buying this device you agree with these conditions.

Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual.

Warranty period for this device is 12 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year.

The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way.

This device will serve you for many years if careful handling.

 **Disposing**

Your product is designed and manufactured with high quality components and materials, recycling devices must take place according to the rules.

This marking on the product, accessories or documents means that at the end of life of the product and its accessories can not be disposed of with other household waste. To avoid harm to the environment or human health from uncontrolled waste disposal, as well as to enable their processing for reuse dispose of the product and its electronic accessories separately from other waste.

Information about the place and method of disposal of the product in accordance with the norms of conservation, contact the retailer or the appropriate government agency. Users should contact their supplier and check the terms of the purchase contract. Observe local regulations. It does not throw the product and its electronic accessories with your other household waste.

Company «Ritmix» has the right to change the Specification without notification.

driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility.

- Test results of alcohol tester are not probative.
- Don't drive in a drunken condition of any degree of intoxication.

Important notes

This device can be used only for personal use. The testing results of this device don't have any validity and can't be used in court. Producer and the seller doesn't have any responsibility if there will be the difference in the testing results between which were made by the third party or by other equipment. By buying this device you agree with these conditions.

Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual.

Warranty period for this device is 12 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year.

The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way.

This device will serve you for many years if careful handling.